



- No pump required
- Cushion is auto-inflating and self adjusting
- Maintenance free
- 100% American made

### Instructions:

- 1 Place AIM Cushion in recliner with straps facing the back of the recliner.
- 2 Feed straps through both sides of the back of recliner.
- 3 Fasten buckle and tighten straps to insure the cushion will not move on seat of recliner.

### Cleaning and Disinfecting

Cleaning and disinfecting are two separate processes. Cleaning must precede disinfection.

**NOTE:** AIM Cushion must be cleaned, disinfected and checked for proper functioning between uses by different individuals.

#### TO CLEAN:

Wipe the cover surface with a neutral detergent and water. Rinse and allow to dry thoroughly.

#### TO DISINFECT:

Wipe the cover surface with dilution of 1 part household bleach to 9 parts water. Follow safety guidelines on bleach container. Rinse and allow to dry thoroughly.

The top cover can occasionally be machine washed in hot water for disinfection. Tumble-dry low. However, the manufacture recommends against repeated washing at high temperatures, as this can accelerate degradation of the cover.